SYNVISC **ONE**®
HYLAN G-F 20
PANAMÁ

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Balazs E.A. (1968). Some aspects of the aging and radiation sensitivity of the intercellular matrix with special regard to hyaluronic acid in synovial fluid and vitreous. In *Thule International Symposium: Aging of Connective and Skeletal Tissue*, October 1-3, 1968, (Eds. Engle, A. and Larsson, T.), Nordiska Bokhandelns Forlag, Stockholm 107-122.

Balazs E.A. and Denlinger, J.L. (1989). Clinical uses of hyaluronan. In *the Biology of Hyaluronan (Ciba Foundation Symposium #143)* (Eds. Evered, D. and Whelan, J.), John Wiley & Sons, Chichester and New York, 265-280.

Balazs E.A. and Denlinger, J.L. (1993). Viscosupplementation: A new concept in the treatment of osteoarthritis. *J. Rheumatol.* 20 (Suppl. 39), 3-9.

Balazs E.A. and Leshchiner, E.A. (1989). Hyaluronan, its crosslinked derivative - hylan - and their medical applications. In *Cellulosics Utilization: Research and Rewards in Cellulosics* (Eds. Inagaki, H. and Philips, G.O.), Elsevier Applied Science, New York, 233-241.

Balazs E.A. and Larsen, N.E. (1997). Safety data from seven clinical studies. *Genzyme Biosurgery Report BXR 9009*.

Band P., Goldman, A., Barbone, A., Reiner, K. and Balazs E.A. (1995). Intra-articular distribution and residence time of hylan polymers. *Materials Research Society, Spring Meeting, April 17-21, 1995, San Francisco, CA, 433 (abstract.)*

Conrozier T., Bertin P., Mathieu P., Charlot J., Bailleul F., Treves R., Vignon E.,

PI: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228344 Annotated: VV-LBL-0228343	PIL ASOCIADO: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228346 Annotated: VV-LBL-0228345
	PIL ASOCIADO: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228348 Annotated: VV-LBL-0228347

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Chevalier X. (2003) Intra- articular injections of hylan G-F 20 in patients with symptomatic hip osteoarthritis: An open-label, multicenter, pilot study. *Clinical Experimental Rheumatology*, 21; 5; 605-610.

Larsen, N.E. and Goldman, A.I. (1997). Residence time of Synvisc® (hylan G-f 20) in the rabbit knee. *Genzyme Biosurgery Report BXR 9010*.

Larsen, N.E. and Lombard, K. M., Parent, E.G. and Balazs E.A. (1992). Effect of hylan on cartilage and chondrocyte cultures. *J. Orthop. Res.* 10, 23-32.

Lussier, A., Cividino, A.A., McFarlane, C.A., Olszynski W.P., Potashner, W.J. and de Médicis, R. (1996). Viscosupplementation with hylan for the treatment of osteoarthritis: findings from clinical practice in Canada. *J. Rheumatol.* 23, 1579-1585.

Peyron J.G. (1993). A new approach to the treatment of osteoarthritis: viscosupplementation. *Osteoarthritis and Cartilage* 1, 85-87.

Pozo, M.A, Balazs, E.A and Belmonte, C. (1997), Reduction of sensory responses to passive movements of inflamed knee joints by hylan, a hyaluronan derivative. *Exp. Brain Res.* 116, 3-9.

Raynuld, J.P., Torrance, G.W. Band, P.A., Goldsmith, CH., Tugwell, P., Walker V., (2002). A prospective randomized, pragmatic, health outcomes trial evaluating the incorporation of hylan G-F 20 into the treatment paradigm for patients with knee osteoarthritis (Part 1 of 2): clinical results. *Osteoarthritis and Cartilage.* 10:506-517.

Scale, D., Wobig, M., and Wolpert, W. (1994). Viscosupplementation of osteoarthritic knees with hylan: a treatment schedule study. *Curr. Ther. Res.* 55, 220-232.

Wobig, M., Dickhut, A., Maier, R. and Vetter, G. (1998.) Viscosupplementation with hylan G-F 20: A week controlled trial of efficacy and safety In the osteoarthritic knee. *Clin. Ther.* 20 (3), 410-42.3

Wobig, M., Bach, G., Beks, P., Dickhut. A, Runzheime, J., Schwiege, G., Vetter, G., and Balazs, E. (1999). The role of elastoviscosity the efficacy of viscosupplementation for osteoarthritis of the knee: A comparison of hylan G-F 20 and lower-molecular-weight hyaluronan. *Clin. Ther.* 21(9), 1549-1562.

Wobig, M., Beks, P., Dickhu,t A., Maier, R., and Vetter G. (1999). Open-label multicenter trial of the safety and efficacy of viscosupplementation with hylan G-F 20 (Synvisc®) in primary osteoarthritis of the knee. *J. Clin. Rheumatol.* 5(6), S24-S31.

<p>PI: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228344 Annotated: VV-LBL-0228343</p>	<p>PIL ASOCIADO: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228346 Annotated: VV-LBL-0228345</p>
	<p>PIL ASOCIADO: PAC_PE_PY_SYNVISCO ONE_INJ_8MGML_PI_L Clean: VV-LBL-0228348 Annotated: VV-LBL-0228347</p>

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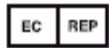
Producto N° / Product number.

Estéril mediante técnica aséptica / Sterile by aseptic technique.

Envase reciclable / Packaging components are recyclable.

No utilizar si el envase está abierto o deteriorado / Do not use if package is damaged or open.

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Intra-articular use
American Industrie

DESCRIPTION

Synvisc (hylan G-F 20) is a sterile, nonpyrogenic, elastoviscous fluid containing hylans. Hylans are derivatives of hyaluronan (sodium salt of hyaluronic acid) and consist of repeating disaccharide units of N-acetylglucosamine and sodium glucuronate. Hylan A has an average molecular weight of approximately 6,000,000 daltons and hylan B is a hydrated gel. Hylan G-F 20 contains hylan A and hylan B (8.0 mg \pm 2.0 mg per ml) in buffered physiological sodium chloride solution (pH 7.2 \pm 0.3).

CHARACTERISTICS

Hylan G-F 20 is biologically similar to hyaluronan. Hyaluronan is a component of synovial fluid which is responsible for its elastoviscosity. The mechanical (elastoviscous) properties of hylan G-F 20 are, however, superior to those of synovial fluid and hyaluronan solutions of comparable concentration. Hylan G-F 20 has an elasticity (storage modulus G') at 2.5 Hz of 111 \pm 13 Pascals (Pa) and a viscosity (loss modulus G'') of 25 \pm 2 Pa. Elasticity and viscosity of knee synovial fluid of 18- to 27-year-old humans measured with comparable method at 2.5 Hz are G' = 117 \pm 13 Pa and G'' = 45 \pm 8 Pa. Hylans are degraded in the body by the same pathway as hyaluronan, and breakdown products are nontoxic.

INDICATIONS AND USAGE

Synvisc (Hylan G-F 20):

- is a temporary replacement and supplement for synovial fluid.
- is beneficial for patients in all stages of joint pathology.
- is most effective in patients who are actively and regularly using the affected joint

Synvisc is only intended for intra-articular use by a physician to treat pain associated with osteoarthritis of the knee.

Achieves its therapeutic effect through viscosupplementation, a process whereby the physiological and rheological states of the arthritic joint tissues are restored.

Viscosupplementation with hylan G-F 20 is a treatment to decrease pain and discomfort, allowing more extensive movement of the joint. In vitro studies have shown that SYNVISC protects cartilage cells against certain physical and chemical damage.

CONTRAINDICATIONS

- Synvisc should not be used in patients with known hypersensitivity (allergy) to hyaluronan (sodium hyaluronate) preparations.
- If venous or lymphatic stasis is present in the relevant limb, Synvisc should not be injected into the joint.
- Synvisc should not be used in infected or severely inflamed joints or in patients having skin diseases or infections in the area of the injection site.

WARNINGS

- Do not inject intravascularly. Intravascular injections may cause systemic adverse events.
- Do not inject SYNVISC extra-articularly or into the synovial tissues and capsule. Adverse events, generally in the area of the injection, have occurred following extra-articular injection of SYNVISC.
- Do not concomitantly use disinfectants containing quaternary ammonium salts for skin preparation because hyaluronan can precipitate in their presence.
- Remove any synovial fluid or effusion prior to injecting Synvisc.

PRECAUTIONS

- Synvisc should not be used if there is a large intra-articular effusion prior to the injection.
- As with any invasive joint procedure, it is recommended that the patient avoid any strenuous activities for approximately 48 hours following the intra-articular injection and resume full activities within a few days.
- Synvisc has not been tested in pregnant women or children under 18 years of age. It is not known if Synvisc is excreted in human milk.
- Synvisc contains small amounts of avian protein and should not be used in patients with related hypersensitivities.
- Do not inject anesthetics or other medications into the knee joint during Synvisc therapy.

ADVERSE EVENTS

Adverse events involving the injected joint that may occur after intra-articular injections of Synvisc are: transient pain and/or, swelling and/or effusion.

In world wide post market experience, injection site reactions have been reported following an intra-articular injection of Synvisc. Injection site events include: pain/tenderness, bruising, swelling, bleeding, itching, redness, rash, warmth.

Postmarketing adverse reactions:

Hypersensitivity reactions including anaphylactic reaction, anaphylactoid reaction, anaphylactic shock and angioedema have been reported.

Cases of acute inflammation, characterized by joint pain, swelling, effusion and sometimes joint warmth and/or stiffness, have been reported following an intra-articular injection of Synvisc. Analysis of synovial fluid reveals aseptic fluid with no crystals. This reaction often responds within a few days to treatment with Non Steroidal Anti Inflammatory Drugs (NSAIDS), intra-articular steroids and/or arthrocentesis.

Clinical benefit from the treatment may still be apparent after such reactions.

DOSAGE GUIDELINES

The dosage regimen for SYNVISC is dependent on the joint being treated.

Osteoarthritis of the knee:

The recommended treatment regimen is three 2 mL injections in the knee, with an interval of 1 week between each injection. To achieve maximum effect, it is essential to administer all three injections. The maximum recommended dosage is six injections within six months, with a minimum of four weeks between treatment regimens.

ADMINISTRATION

- Do not use SYNVISC if package is opened or damaged.
- The contents of the syringe must be used immediately after its packaging is opened.
- Remove synovial fluid or effusion before injecting SYNVISC.
- Inject at room temperature.
- To remove the syringe from the blister (or tray); take hold of it by the body, without touching the plunger rod.
- Administer using strict aseptic procedures, taking particular care when removing the tip cap.
- Twist the tip cap before pulling it off, as this will minimize product leakage
- Use an appropriate size of needle depending on the joint to be treated, 18 to 22 gauge for SYNVISC.
- To ensure a tight seal and prevent leakage during administration secure the needle tightly while firmly holding the Luer hub.
- Do not tighten or apply excessive leverage when attaching the needle or removing the needle guard, as this may break the tip of the syringe.
- Inject into the synovial space only, using if necessary, appropriate guidance such as fluoroscopy especially in joints such as the hip and shoulder.
- When using fluoroscopic guidance, ionic or non-ionic contrast agent may be utilized. No more than 1 ml of contrast agent should be used per 2 ml of hylan G-F 20.

- The syringe contents are for single use only. The recommended dosage guidelines state to inject the full volume (2 mL for Synvisc) of the syringe.
- Do not re-use the syringe and/or needle. Re-use of syringes, needles and/or product from a used syringe may result in loss of sterility, product contamination and/or incomplete treatment
- Do not re-sterilise hylan G-F 20.

DURATION OF EFFECT

Generally, the duration of effect for those patients who respond to treatment has been reported up to 26 weeks, although shorter and longer periods have also been observed.

However, prospective clinical data in knee osteoarthritis patients have shown benefit of treatment up to 52 weeks, following a single course of three SYNVISC injections.

CONTENT PER 1 mL

Each 1 ml of Synvisc contains: 8.0 mg of Hylan G-F 20, sodium chloride, disodium hydrogen phosphate, sodium dihydrogen phosphate hydrate, water for injection q.s.

HOW SUPPLIED

Synvisc is supplied in a 2.25 ml glass syringe containing 2 ml hylan G-F 20.

The contents of each syringe are sterile and nonpyrogenic.

STORAGE CONDITIONS:

Store between 2°C - 30°C. Do not freeze

Manufactured in United States by:

Genzyme Corporation

1125 Pleasant View Terrace Ridgefield, NJ 07657, USA

A division of Genzyme Corporation

Cambridge, MA 02142, USA

Imported / Distributed:

Imported and Distributed in Venezuela by sanofi-aventis de Venezuela S.A. RIF J-30308848-1. CPE 0413303529. Reg. San .: E.F. 30146. Farm. Sponsor: Dra. Sandra Carrillo. With Optional Prescription.

sanofi-aventis de Guatemala S.A. Reg. San .: PMQ-301

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Synvisc y Genzyme son marcas registradas, y Synvisc-One es una marca comercial de Genzyme Corporation / Synvisc and Genzyme are registered trademarks, and Synvisc-One is a trademark of Genzyme Corporation.