Toujeo® SoloStar®

insulin glargine 300 units/mL

Six simple steps for injecting

Step

1

Check your pen

Always check the name and expiry date on the label to ensure

- ✓ You have the correct insulin
- ✓ The expiry date has not passed

Pull off the pen cap and check that the insulin is clear

- ✗ Do not use the pen if the insulin looks cloudy, colored or contains particles


! Remember to remove your new pen from the fridge at least one hour before you inject

Step

2

Attach a new needle

- ✓ Always use a new needle. This helps stop blocked needles, contamination and infection
- ✓ Always use the needle that is compatible with the use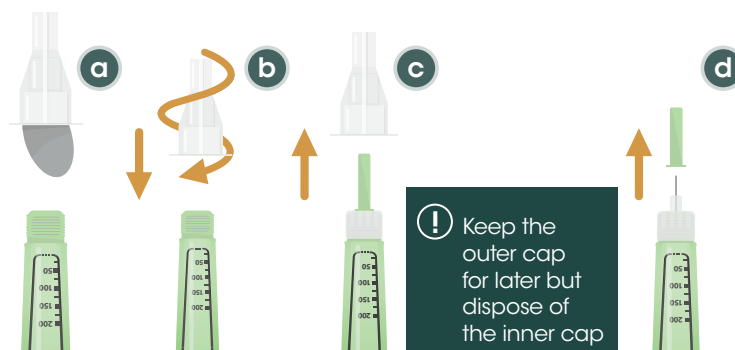


! Keep the outer cap for later but dispose of the inner cap

Step

3

Perform a safety test

- ✓ Always perform a safety test before each injection to ensure your pen and needle are working properly
- ✓ Select and dispense 3 units of insulin
- ✓ If no insulin comes out of the needle tip, you may need to repeat the safety test 3 times before seeing insulin
- ✓ If no insulin appears after the third test, change the needle and repeat the test. If there is still no insulin, use a new pen

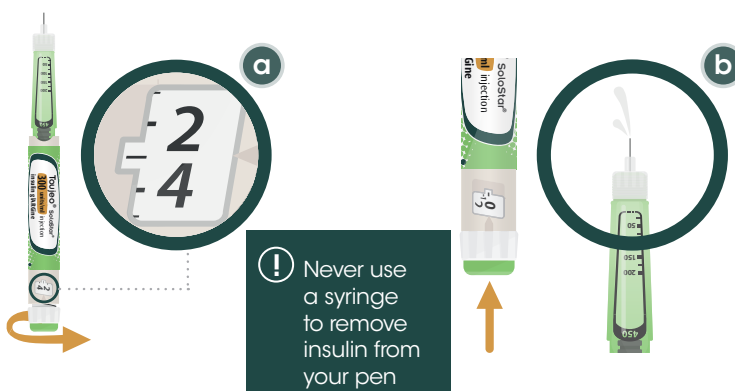! Never use a syringe to remove insulin from your pen

Step

4

Select the dose

- ✓ Make sure the needle is attached and the dose is set to "0"
- ✓ Turn the dose selector until the dose pointer lines up with your intended dose
- ✓ If you turn past your dose, you can turn back down
- ✓ If you cannot select your full prescribed dose, split the dose into 2 injections or use a new pen
- ✗ Never select a dose or press the injection button without a needle attached. This may damage your pen.

! A maximum of 80 units can be delivered per injection

Toujeo® SoloStar®

insulin glargine 300 units/mL


Six simple steps for injecting

Step

5

Inject your dose

- ✓ Inject Toujeo® SoloStar® into your stomach, upper arms or thighs subcutaneously
- ✓ Push the needle into your skin, as shown by your doctor, pharmacist or nurse
- ✓ Place your thumb on the injection button and then press all the way in and hold
- ✓ Keep the injection button held in and when you see "0" in the dose window, slowly count from 1 to 5 before releasing the button and removing the needle from your skin. This will make sure you get your full dose
- ✗ If you find it hard to press the injection button in, do not force it as this may break your pen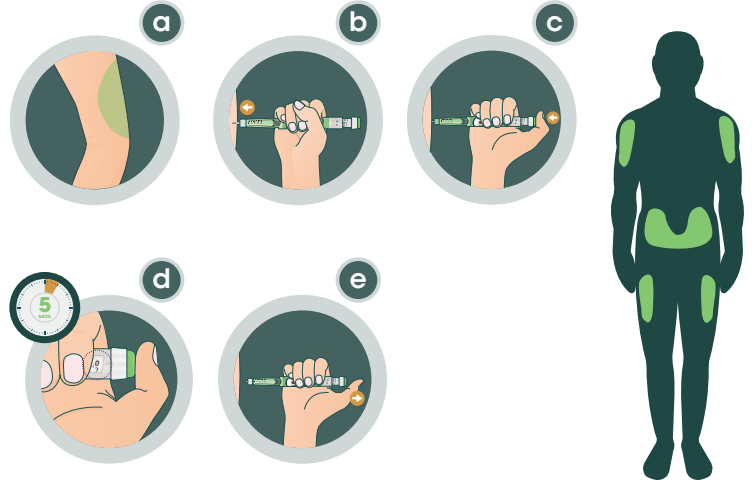


Step

6

Remove and dispose of the needle

- ✓ Take care when handling needles, this is to prevent injury and infection
- ✓ Throw away the used needle in a puncture resistant container
- ✗ Never put the inner needle cap back on

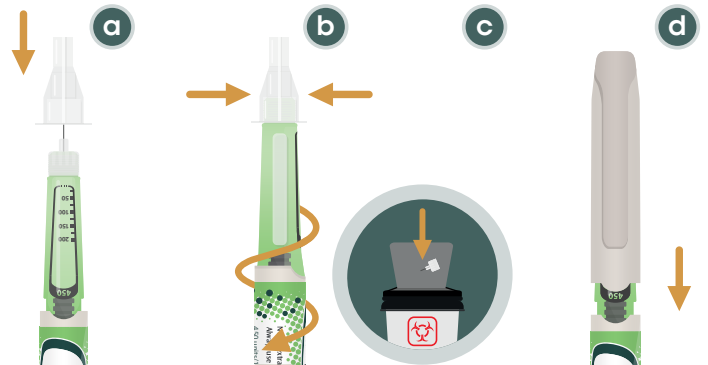Remember

- ✓ When using your pen for the first time, remove it from the fridge at least an hour before first use. Avoid injecting insulin straight out of the fridge. This will feel more comfortable
- ✓ After first use keep your pen at room temperature below 30°C and never put your pen back in the fridge
- ✓ Always carry a spare pen and spare needles in case they get lost or damaged
- ✓ Always use a new needle each time you use the pen
- ✓ Only use your pen for up to 6 weeks after its first use
- ✗ Never share your pen with anyone else, it is meant for you alone
- ✗ Never attempt to remove the insulin with a syringe from your pen

SANOFI, Kingdom of Saudi Arabia, P.O. Box 9874, Jeddah 21423, K.SA Tel: +966-12-669-33181 Fax: +966-12-663-6191
For Medical Information, Please contact: +966-12-669-3318, ksa_medicalinformation@sanofi.com,
For Pharmacovigilance, Please contact: +966-544-284-797, ksa_pharmacovigilance@sanofi.com
To report any Product Technical Complaint, please contact SANOFI Quality Department: Email: KSA_PTC_Reporting@sanofi.com
www.sanofi.com.sa

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